


ELIMINATE™

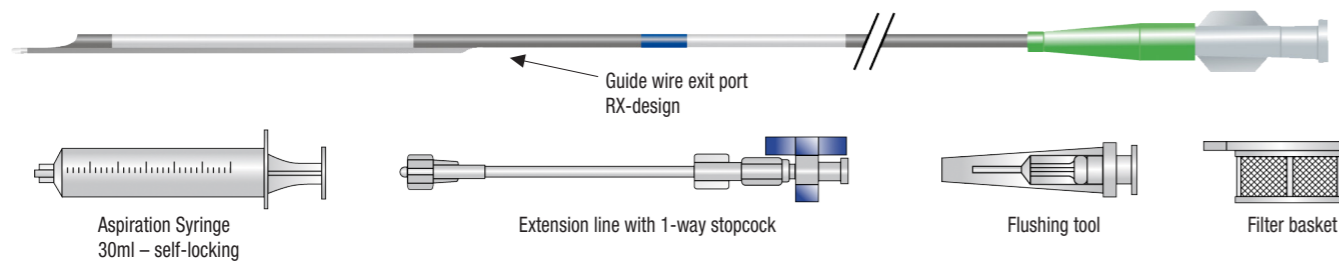

ELIMINATE™

Aspiration Catheter

Maximize aspiration performance
Thrombus aspiration catheter **8Fr**

Product code	Guiding Catheter Compatibility	Guide Sheath Compatibility	Outer diameter Distal/Proximal)	Aspiration lumen area Distal/Proximal)	RX segment length	Catheter length
EG1401	8Fr I.D.≥0.086" (2.18mm)	6Fr	2.15mm (6.5Fr)/ 1.76mm (5.3Fr)	1.58mm ² / 1.65mm ²	23cm	140cm
EG1652	7Fr I.D.≥0.080" (2.03mm)	-	1.96mm (5.9Fr)/ 1.60mm (4.8Fr)	1.23mm ² / 1.33mm ²	23cm	140cm
EG1602	6Fr I.D.≥0.070" (1.78mm)	-	1.70mm (5.2Fr)/ 1.40mm (4.2Fr)	0.78mm ² / 0.95mm ²	23cm	140cm

Guide wire compatibility 0.014" (0.36 mm)
Product codes EG1652 and EG1602 contain a mandrel for insertion of the aspiration catheter



Terumo Corporation
44-1, 2-chome, Hatagaya
Shibuya-ku, Tokyo 151-0072,
Japan
Tel.: +81 3 3374 8111
Fax: +81 3 3374 8196

European Headquarters

Terumo Europe N.V.
Interleuvenlaan 40
3001 Leuven, Belgium
Tel.: +32 16 38 12 11
Fax: +32 16 40 02 49

**Terumo Europe
Interventional Systems
Liaison Office**
Immeuble Krebs
8, rue Jean-Pierre Timbaud
CS 10720
78067 Saint Quentin
en Yvelines Cedex,
France
Tel.: +33 139 30 95 75
Fax: +33 139 30 95 70

European Offices

**Terumo Europe N.V.
Benelux Sales Division**
Interleuvenlaan 40
3001 Leuven, Belgium
Tel.: +32 16 39 25 80
Fax: +32 16 39 25 99
The Netherlands:
Tel.: 0800 0220396
Fax: 0800 0220414

Laboratoires Terumo France S.A.
Parc Ariane - Bât. Uranus
1, rue Hélène Boucher
78284 Guyancourt Cedex, France
Tel.: +33 130 96 13 00
Fax: +33 130 43 60 85

Terumo Italia S.r.l.
Via Simone Martini, 143/145
00142 Roma RM, Italy
Tel.: +39 0651 96 14 20
Fax: +39 0650 30 407

Terumo UK Ltd
Tamesis, The Causeway
Egham, Surrey
TW20 9AW, United Kingdom
Tel.: +44 1784 476 200
Fax: +44 1784 476 201

Terumo Deutschland GmbH
Ludwig-Erhard-Strasse 6
65760 Eschborn, Germany
Tel.: +49 6196 80 230
Fax: +49 6196 80 23 200

**Terumo Deutschland GmbH
Zweigniederlassung Österreich**
Liebermannstrasse F10 301
2345 Brunn am Gebirge, Österreich
Tel.: +43 2236 378 020
Fax: +43 2236 378 020 100

Terumo Deutschland GmbH
Zweigniederlassung Spreitenbach
Bodenackerstrasse 3
8957 Spreitenbach, Switzerland
Tel.: +41 56 419 10 10
Fax: +41 56 419 10 11

Terumo Europe España SL
Avda. Juan Carlos I, Nº 13 - 7ª Planta
Edificio Torre La Garena
28806 Alcalá de Henares (Madrid), Spain
Tel.: +34 9021 01 298
Fax: +34 9021 01 358

**Terumo Denmark
Filial of Terumo Sweden AB**
Tel.: +45 7020 93 80
Fax: +45 7020 94 80

Terumo Sweden AB
Nya Varvet, Byggnad 90
Sven Källfelts Gata 18
426 71 Västra Frölunda, Sweden
Tel.: +46 3174 85 880
Fax: +46 3174 85 890

Terumo Russia LLC
BC Northern Tower
13 Floor, 10 Testovskaya Street
Moscow 123317, Russia
Tel.: +7 495 988 4740
Fax: +7 495 988 4739

**Terumo Corporation Dubai Branch
(Middle East and Africa)**
Al Masraf Tower, 22nd Floor
P. O. Box 20291 Dubai
U.A.E.
Tel.: +971 4 221 2220
Fax: +971 4 221 3330

*Addresses subject to change,
please consult our Website.*


© Registered Trademark
Published by Terumo Europe NV

IS414GB0519XW0513E

3 steps for successful thrombus removal


Fully usable 5.3Fr aspiration lumen with separate 0.014" (0.36mm) guide wire lumen

1 Prepare

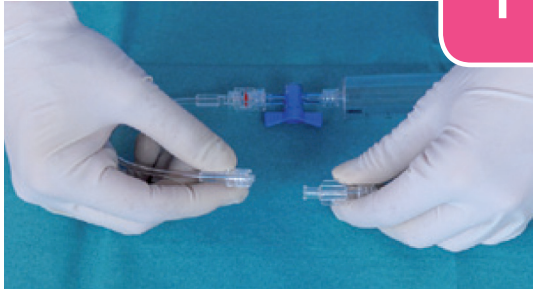


Attach the pre-connected extension set to the Eliminate aspiration catheter and flush.

2 Deliver

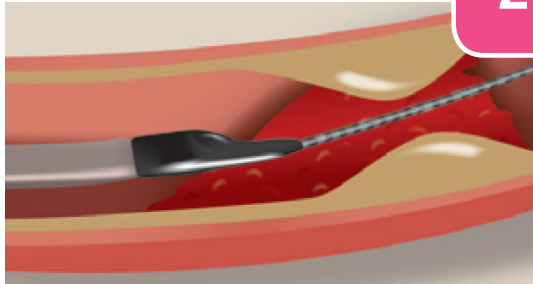


Backload Eliminate aspiration catheter onto the guide wire and advance the catheter under fluoroscopic guidance until the radiopaque marker band is proximal to the thrombus.

3 Aspirate



Close the stopcock valve and create a vacuum by pulling back the plunger to the desired volume.






Open the stopcock and slowly advance the Eliminate aspiration catheter towards the thrombus.

Remove the catheter from the patient while maintaining the vacuum /negative pressure via the syringe.


Repeat the aspiration procedure if needed

For maximal aspiration performance while leaving a guide wire across the lesion

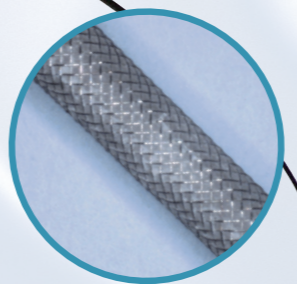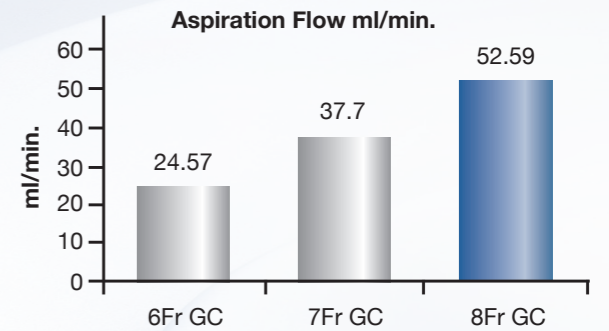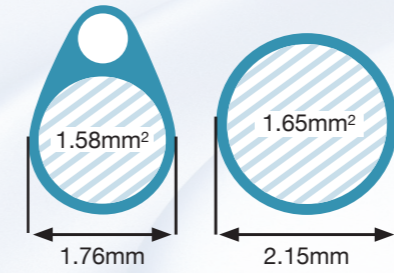

Braided shaft

For superior trackability and kink-resistance

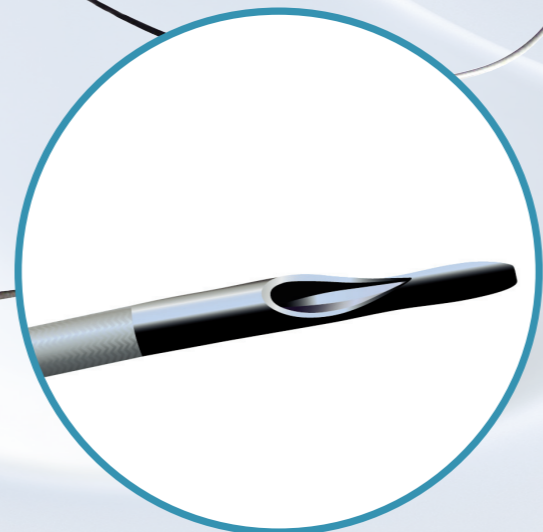

Dedicated beveled and rounded tip design

For an improved, atraumatic crossability and effective aspiration capabilities

Kit package

- All you need in one packaging
 - 2x 30ml syringes
 - 1 filter basket
 - 1x extension line with stopcock
 - 1 flushing tool

